MOPEDIA®

ITALIANO

MA RS998-RS999 01 A_ITA_09-2020

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MANUALE D'ISTRUZIONE

INDICE

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MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com **MADE IN TAIWAN**

MOPEDIA®

ENGLISH

MA RS998-RS999 01 A_ENG_09-2020

OVERBED TABLES

INSTRUCTION MANUAL

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 View and List of Components.....	pag. 5
8. ASSEMBLING/DISASSEMBLING	PAG.6
8.1 Assembling.....	pag. 6
8.2 Disassembling	pag. 7
9. BEFORE USE	PAG.7
10. PRECAUTIONS FOR USE	PAG.7
11. HOW TO USE.....	PAG.7
12. MAINTENANCE	PAG.7
13. CLEANING AND DISINFECTION	PAG.7
13.1 Cleaning.....	pag. 7
13.2 Disinfection.....	pag. 8
14. CONDITIONS OF DISPOSAL	PAG.8
15. SPARE PARTS.....	PAG.8
16. TECHNICAL FEATURES.....	PAG.8
16.1 Dimensions and weight	pag. 8
16.2 Technical specifications.....	pag. 8
17. WARRANTY.....	PAG.9
18. REPAIRING.....	PAG.9
19. SPARE PARTS	PAG.9
20. EXEMPT CLAUSES	PAG.9

I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS998 Height adjustable overbed table with castors

RS999 Tilting height adjustable overbed table with castors, two tops

2. INTRODUCTION

Thank you for choosing an overbed table by MOPEDIA range, manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Overbed tables by MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the tables you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the overbed table. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The MOPEDIA bed trays are primarily intended for people who have mobility problems, mainly bedridden where it is expected the consumption of meals in bed or chair. However they are adaptable both in hospitals and home.

NOTE:

Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of OVERBED TABLES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.
- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Medical Device

Manufacturer

Read the instruction manual

Batch Lot

Conditions of disposal


7. GENERAL DESCRIPTION

7.1 View and List of Components

RS998 - Height adjustable overbed table

Parts list:


- nr.1 four castors (ø 50mm) powder coating C shaped steel frame;
- nr.1 powder coating steel column with hydraulic stepless piston and locking pin;
- nr.1 MDF top with plate for assembly underneath;
- nr. 5 M8 screws;
- nr. 1 M8 exagonal wrench;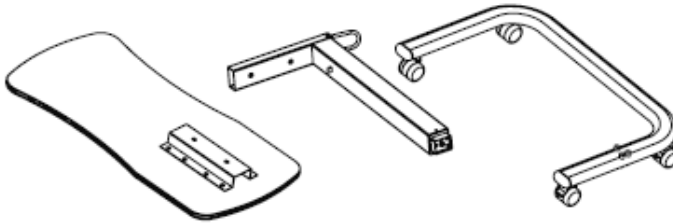

RS999 - Tilting height adjustable overbed table, two tops

Parts list:

- nr.1 four castors (ø 50mm) powder coating c shaped steel frame;
- nr.1 powder coating steel column with hydraulic stepless piston and locking pin;
- nr.1 MDF top with plate for assembly underneath;
- nr. 5 M8 screws;
- nr. 1 M8 exagonal wrench;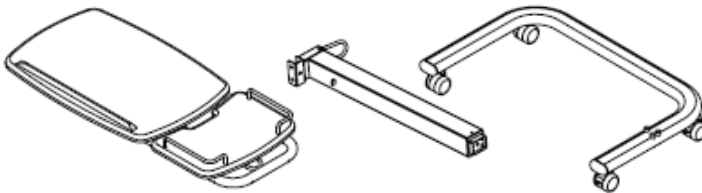


8. ASSEMBLING/DISASSEMBLING

8.1 Assembling


RS998 - Height adjustable overbed table with castors

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Place the top overturned on a leveled plane, position column bracket above table top bracket until holes match. Use two of five screws (the shorter) and the M8 exagonal wrench to assembly the parts (rif. A).
3. Place also the C shaped frame overturned until holes match with those placed on the bottom base of the column. Use the remaining screws and the M8 exagonal wrench to assembly the parts (rif. B).
4. Flip 180° the product now assembled and remove the locking pin (rif. C). Apply a force on the lower side of the top to raise the table (rif. D), and apply the force on the upper side of the top, while switching up the lever, to lower the table (rif. E).
5. Now the table is ready to be used.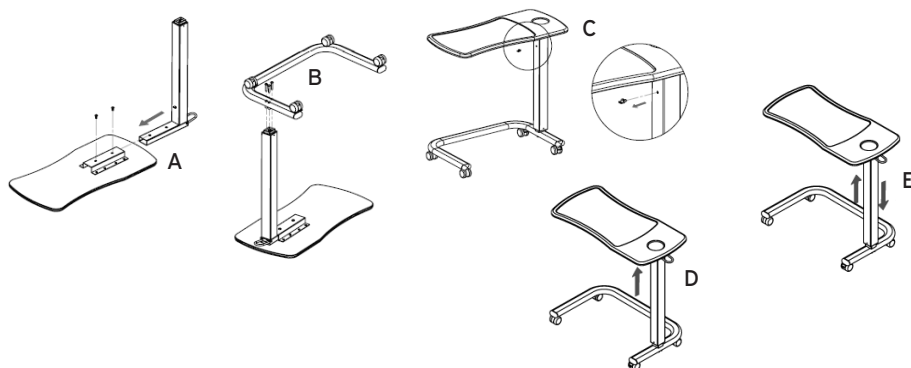

RS999 - Tilting height adjustable table with castors, two tops

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Place the top overturned on a leveled plane, position the column c shaped plate above square tube of the table until holes match. Use two of five screws (the shorter) and the M8 exagonal wrench to assembly the parts (rif. F).
3. Place also the C shaped frame overturned until holes match with those placed on the bottom base of the column. Use the remaining screws and the M8 exagonal wrench to assembly the parts (rif. G).
4. Flip 180° the product now assembled and remove the locking pin (rif. H). Apply a force on the lower side of the top to raise the table (rif. I), and apply the force on the upper side of the top, while switching up the lever, to lower the table (rif. L).
5. Now the table is ready to be used.

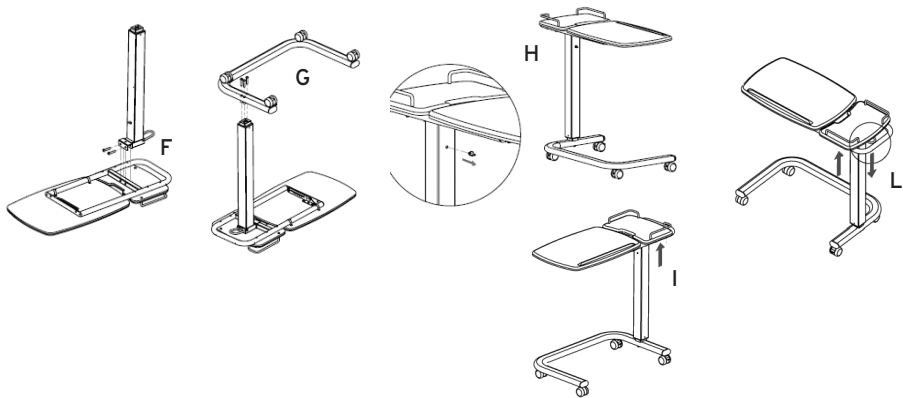8.2 Disassembling

To disassemble do the exact opposite of the above instructions.

9. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the knobs.

10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make sure that the floor surface supporting the overbed table is level.
- Make the adjustment operations without load.
- Check all components are well fixed.
- Always assist the patient during the elevation operation

11. HOW TO USE

RS998 and RS999

Pull upwards from the bottom of the top to raise the table.

Push downwards from the upper side of the top, while switching up the lever, placed on the upper side of the column, to lower the table.

12. MAINTENANCE

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and / or terminals.

13. CLEANING AND DISINFECTION

13.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

14. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal recycling areas for the expected operations of recycling the materials used.

15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue. ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

16. TECHNICAL FEATURES

16.1 Dimensions and weight

RS998

Height:	79-114 cm
C shaped frame dimensions:	71*45 cm (W*D)
Top dimensions:	80*39 cm (W*D)
Net weight:	13 kg
Max. load:	25 kg

RS999

Height:	79-114 cm
C shaped frame dimensions:	71*45 cm (W*D)
Tops dimensions:	small top 25*35 cm (W*D), big top 57*39 cm (W*D)
Net weight:	16 kg
Max. load:	25 kg

16.2 Technical specifications

RS998

- C shaped powder coating steel frame with four ø 50 mm castors
- Powder coating steel column with hydraulic piston
- Anti-scratch MDF top

RS999

- C shaped powder coating steel frame with four ø 50 mm castors
- Powder coating steel column with hydraulic piston
- Anti-scratch MDF small top, with fall arrest iron rod on both sides
- Anti-scratch MDF big top with fall arrest edge on one side

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPAIRING

Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonérations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

**WARRANTY CERTIFICATE**

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN TAIWAN**

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