FERMACAPO – UNIVERSAL HEAD IMMOBILIZER - INMOVILIZADOR UNIVERSAL

ITALIANO

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ENGLISH

INDICATIONS : FERMACAPO is designed to aid immobilization of a patient's head and neck during transportation when injury is suspected. The head immobilizer can be used on Spinal Board NFC System foldable stretchers foldable in two and SCOOP stretcher. Minimal interference with X-ray

SPECIFICATIONS : FERMACAPO is compound of a Base plate fixed on the stretcher with two + one side retentionstraps secured with the VELCRO ; two Support pads to immobilize patient's head and two Strap pads to fix forehead and chin (see fig. 1). FERMACAPO is realized with a soft material covered with a special washable vinyl paint

INSTRUCTIONS : secure the head immobilizer Base plate to the stretcher (see fig. 2). Using EMS transfer techniques, move the patient onto the stretcher ; place the straight side of each Support pad against each side of the patient's head (see fig 3) and place forehead/chin straps over the patient securing with VELCRO (see fig. 4). In cases of suspected head and/or neck injury a cervical collar should be applied prior to moving the patient.

CAUTION : should be applied by skilled staff

CLEANING AND MAINTENANCE INSTRUCTIONS : after using verify integrity of FERMACAPO ; eventually replace damaged parts. Clean the head immobilizer with warm water and soap

STORAGE : should be kept in a dry room, dust free . Storage as a compact unit .

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