**MOPEDIA®**

## **PORTABOMBOLA IN TESSUTO**

---

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## INDICE

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**Class I Medical Equipment**  
 according Directive 93/42/EEC and subsequent amendment

## 1. CODES

**RCA136** Oxygen bottle holder

**CPA606** Oxygen bottle holder

## 2. INTRODUCTION

Thank you for choosing an accessories manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. The accessory by MORETTI S.p.A. is designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the accessory you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the accessory. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

The **Oxygen bottle holder** is primarily intended for people who have mobility and respiratory problems. However they are adaptable both in hospitals and home.

**NOTE:** Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions

### WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to wheelchairs and aid accessories range are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/ EEC and further modifications. For this purpose Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive.
2. The full list of the devices at issue are specified in Annex A in declaration.
3. The devices in question ARE NOT MEASUREMENT DEVICES.
4. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
5. The devices in question are sold in NON STERILE packing.
6. The devices in question should be considered as belonging to class I.
7. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC and further modifications to the proper authorities.

### 4.1 Standards and guidelines reference

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## 5. GENERAL WARNINGS

### WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.

## 6. GENERAL DESCRIPTION

### 6.1 View and List of Components

The Oxygen bottle holder is provided as a single piece, ready to use.

## 7. ASSEMBLING/DISASSEMBLING

### 7.1 Assembling

 Max diameter Oxygen bottle: 12 cm

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Fix the upper belts on the back of the wheelchair or chair by using the clips.
3. Fix the lower belts on the frame of the wheelchair or chair by using the clips.
4. Place the Oxygen bottle inside the Oxygen bottle holder.
5. Now the Oxygen bottle holder is ready for use.

RCA136

CPA606

### 7.2 Disassembling

To disassemble do the exact opposite of the above instructions.

## 8. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the belt.

## 9. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make the adjustment operations without Oxygen bottle.
- Check all components are well fixed.

## 10. HOW TO USE

The assistant must place and adjust the Oxygen bottle holder with the maximum care to avoid sudden movements or dangerous oscillations. An assistant has to check any movement of the patient.

The Oxygen bottle holder is ready to use and will be sufficient to pull out the belts and fix it on the backrest and the frame of the wheelchair or commode chair through the appropriate clips.

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## 12. CLEANING AND DISINFECTION

### 12.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

#### **WARNING!**

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

If you need, Use machine wash with a gentle cycle or hand wash 30°C with neutral soaps or detergents and delicate.

#### PRECAUTIONS:

- Do not Bleach
- Do not dry clean
- Dry Flat away from direct sunlight
- Tumble dry low
- Do not iron
- Use non-enzyme detergent
- Do not wash with other colors

### 12.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent..

## 13. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

## 14. SPARE PARTS


For spare parts and accessories, please always refer to the MORETTI master catalogue.  
**ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.**

## 15. TECHNICAL FEATURES

### 15.1 Dimension

**RCA136** Oxygen bottle holder

**CPA606** Oxygen bottle holder

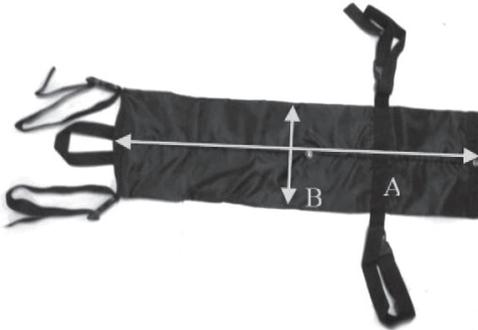Height "A": 68 cm

Width "B": 19 cm

Oxygen bottle MAX diameter: 12 cm

Pay load : 20 kg

### 15.2 Technical specifications

- Polyester tissue
- PP belts
- Black Color

## 16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual.

The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

## 17. REPAIRING

### Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

**Repair not covered by warranty**

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

**Non defective device**

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

**18. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

**19. EXEMPT CLAUSES**

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

**WARRANTY CERTIFICATE**
**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com email: info@morettispa.com**
**MADE IN P.R.C.**

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)