PORTABOMBOLA E ASTA FLEBO

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INDICE

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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com

OXYGEN BOTTLE HOLDER AND I.V.POLE

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INDEX

1. CODES	pag. 3
2. INTRODUCTION	pag. 3
3. INTENDED USE	pag. 3
4. EC DECLARATION OF CONFORMITY	pag. 3
4.1 Standards and guidelines reference	pag. 3
5. GENERAL WARNINGS.....	pag. 4
6. GENERAL DESCRIPTION.....	pag. 4
6.1 View and List of Components CPA600 - I.V. pole	pag. 4
6.2 View and List of Components CPA605 - Metal Oxygen bottle holder.....	pag. 4
7. ASSEMBLING/DISASSEMBLING	pag. 5
7.1 Assembling CPA600 - I.V. pole	pag. 5
7.2 Assembling CPA605 - Metal Oxygen bottle holder	pag. 5
7.3 Disassembling.....	pag. 5
8. BEFORE USE.....	pag. 5
9. PRECAUTIONS FOR USE.....	pag. 5
10. HOW TO USE	pag. 6
11. MAINTENANCE	pag. 6
12. CLEANING AND DISINFECTION.....	pag. 6
12.1 Cleaning.....	pag. 6
12.2 Disinfection	pag. 6
13. CONDITIONS OF DISPOSAL	pag. 6
14. SPARE PARTS	pag. 6
15. TECHNICAL FEATURES	pag. 6
15.1 Dimension	pag. 6
15.2 Technical specifications	pag. 7
16. WARRANTY	pag. 7
17. REPAIRING.....	pag. 7
18. SPARE PARTS	pag. 7
19. EXEMPT CLAUSES	pag. 8

CE Class I Medical Equipment
according Directive 93/42/EEC and subsequent amendment

1. CODES

CPA600 I.V. pole

CPA605 Metal Oxygen bottle holder

2. INTRODUCTION

Thank you for choosing an accessories manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. The accessory by MORETTI S.p.A. is designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the accessory you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the accessory. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The **Oxygen bottle holder** and **I.V. pole** is primarily intended for people who have mobility and respiratory problems. However they are adaptable both in hospitals and home.

NOTE: Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to wheelchairs accessories range are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. For this purpose Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive.
2. The full list of the devices at issue are specified in Annex A in declaration.
3. The devices in question ARE NOT MEASUREMENT DEVICES.
4. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
5. The devices in question are sold in NON STERILE packing.
6. The devices in question should be considered as belonging to class I.
7. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC and further modifications to the proper authorities.

4.1 Standards and guidelines reference

Safety tests required by UNI EN 12182

5. GENERAL WARNINGS

WARNING!


Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.

6. GENERAL DESCRIPTION


6.1 View and List of Components CPA600 - I.V. pole

1. Fixing tube
2. Fixing supports
3. Telescopic tube
4. Knob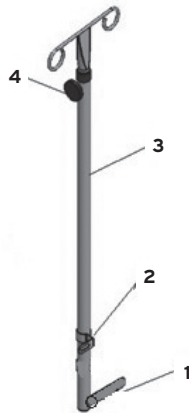

6.2 View and List of Components CPA605 - Metal Oxygen bottle holder

1. Slidings Plates Fixing (4pcs)
2. Knob(2pcs)
3. Frame

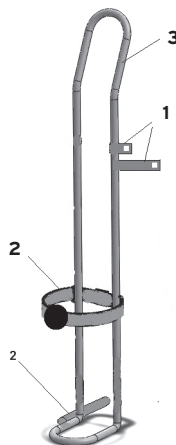7. ASSEMBLING/DISASSEMBLING

7.1 Assembling CPA600 - I.V. pole

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Insert the fixing tube "1" on the bottom frame on the wheelchair.
3. Fix the IV pole by the support "2" on the backrest using the existing screws of the wheelchair.
4. Now the accessory is ready for use.

Picture with illustrative purpose only

7.2 Assembling CPA605 - Metal Oxygen bottle holder

WARNING!

Make the adjustment operations without of the oxygen bottle.

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Insert the fixing tube "2" on the bottom frame on the wheelchair.
3. Fix the Oxygen bottle holder by the support "1" on wheelchair frame using the existing screws of the wheelchair and the Oxygen bottle holder.
4. **TIGHTEN THE SCREWS** of the supports.
5. Make sure that everything is fixed in your aid.
6. Now the accessory is ready for use.

Picture with illustrative purpose only

7.3 Disassembling

To disassemble do the exact opposite of the above instructions.

8. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the belt

9. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make the adjustment operations without Oxygen bottle.
- Check all components are well fixed.

10. HOW TO USE

The assistant must place and adjust the accessory with the maximum care to avoid sudden movements or dangerous oscillations by the hooks. An assistant has to check any movement of the patient.

11. MAINTENANCE

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and/or terminals.

12. CLEANING AND DISINFECTION

12.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

WARNING!

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

12.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

13. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

14. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue. ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

15. TECHNICAL FEATURES

15.1 Dimension

CPA600 - I.V. POLE

Height "A": 100cm

Width "B": 22cm

Pay load: 5 kg

CPA605 - Metal Oxygen bottle holder

Height "A": 78cm

Width "B": 15cm

Oxygen bottle MAX diameter: 15cm

Pay load: 20 kg circa

15.2 Technical specifications

- Chromed plated steel

The dimensions maybe change $\pm 0,5\text{cm}$

16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

17. REPAIRING

Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

19. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____ **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com