# User and maintenance manual

*jamme 'o bagno*

**joy on the beach**

**NEATECH™**

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under its responsibility,  
states that the wheelchair

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satisfies the conditions laid down  
by European Directive 93/42;

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according to the criteria for  
classification of annex IX of this  
Directive, the Job is classified as

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### **DISPOSING**

This product and all its components can not be treated as household waste. For more detailed information on how recycle this product contact your local waste disposal service.

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Dear Friend,

Thank You for choosing our product.

The JOB is a chair for the transport of elderly and disabled people adapted to the sea, snow or off-road trekking. It is provided with a pair of wheels designed for an easy transport on all type of surface (sand, pebbles, snow).

Because of the construction with aluminum alloy materials that don't fear the attack of the salt, with the Job you can enter the water staying comfortably seated.

It is completely demountable to facilitate the transport also in small vehicles.

## Features

- 44 cm diameter wheels with tires extractable
- Completely disassembling
- Reduced overall dimensions of the frame disassembled
- Aluminum frame
- Galvanic treatment and epoxy powder painting

**WARNING:** It is prohibited to use the Wheelchair or its parts for any purpose other than that indicated; for a correct use please follow the instructions given in this manual. **NEATECH.IT disclaims any responsibility for damages caused by improper use of aids.**

**WARNING:** The information contained in this manual may be subject to change and without prior notice.

## 2. Starting up

### 2.1. Checks to be made on delivery

- Check for the integrity of the original packaging.
- Check for any anomalies on the shipping documents.
- Check for the functionality and integrity of the device in all its parts, at the time of delivery or immediately thereafter, to ensure that no damage has resulted from a careless transport.
- Make sure the surface of the device is not damaged, scratched, bent, etc.

### 2.2. Unpacking

#### **Inside the box there is:**

- Backrest with upholstery
- Footrest with upholstery
- Front support
- Anti-tip bar
- Main frame with upholstery and tube fasteners
- N. 2 Pneumatic wheels
- N.2 wheel fasteners
- Documentation
- Purchased accessories

#### **PACKAGING DIASPOSAL**

To properly recycle the packaging materials follow the instructions provided by your local waste disposal service.

## 2.3. Mounting the wheelchair

### Anti-tip bar and front support bar

Insert anti-tip bar (D) and front support bar (C) into the main chassis (E) as shown in Figure 29. Secure the two bars with fasteners.

Make sure tubes are correctly locked.

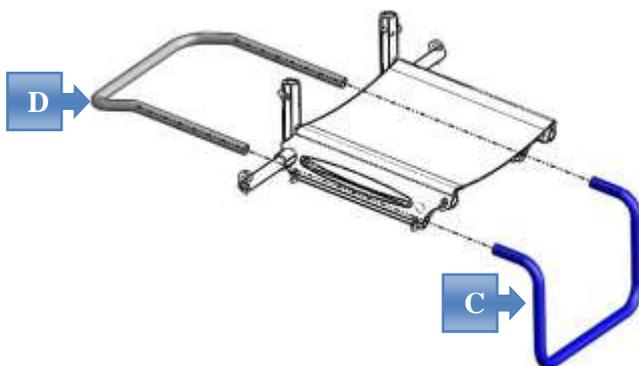
Figure 29

#### **WARNING**

Tubes (D) and (C) are of the same width but (D) is longer.

If tubes (D) and (C) are inverted, the wheelchair will result unbalanced with possible risk for the user.

## Wheels

Insert the wheel (lh and rh) on the axis of the chassis as shown in Figure 30.

Use the special housing hole on the hub, previously aligned with the hole present on the axle of the frame and insert the fastener (F)

Make sure each wheel fastener is correctly locked on hub.


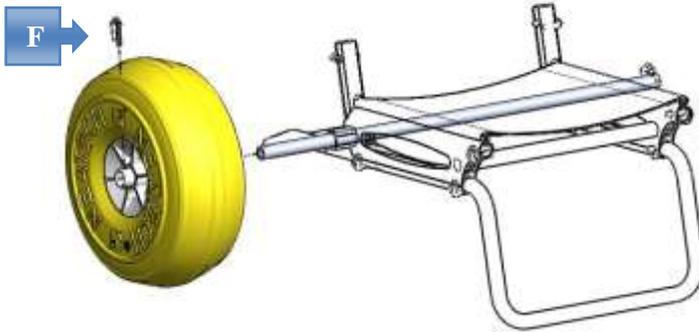
Figure 30

## Backrest


Insert the backrest into the bushings on the main frame as shown in Figure 31, then insert the 2 fasteners deeply through proper holes to lock it.

Make sure fasteners are correctly locked on main frame.

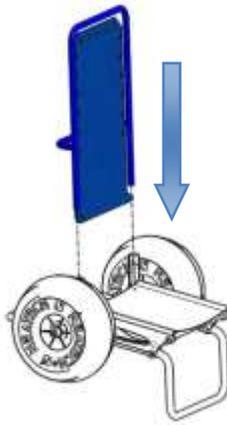**Figure 31**

After that fix strip located on bottom end of the backrest, passing it around wheels' axle as shown in Figure 32.

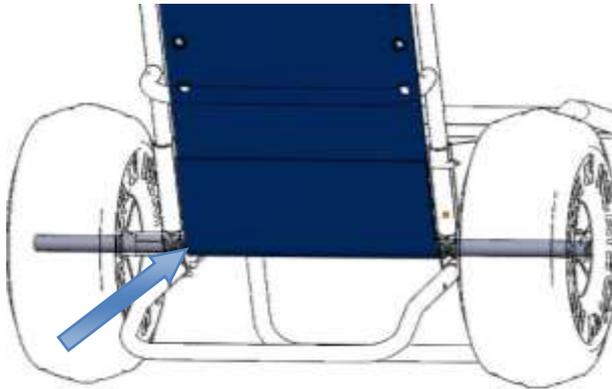**Figure 32**

**WARNING**

It is very important to carefully fix the backrest with the fixing clips and the strip to avoid possible dismounting

## Leg bar

Insert the legbar in the tubes of the chassis as shown in Figure 33 and lock it with the 2 fasteners (G). For adjustment instructions see section 3.1

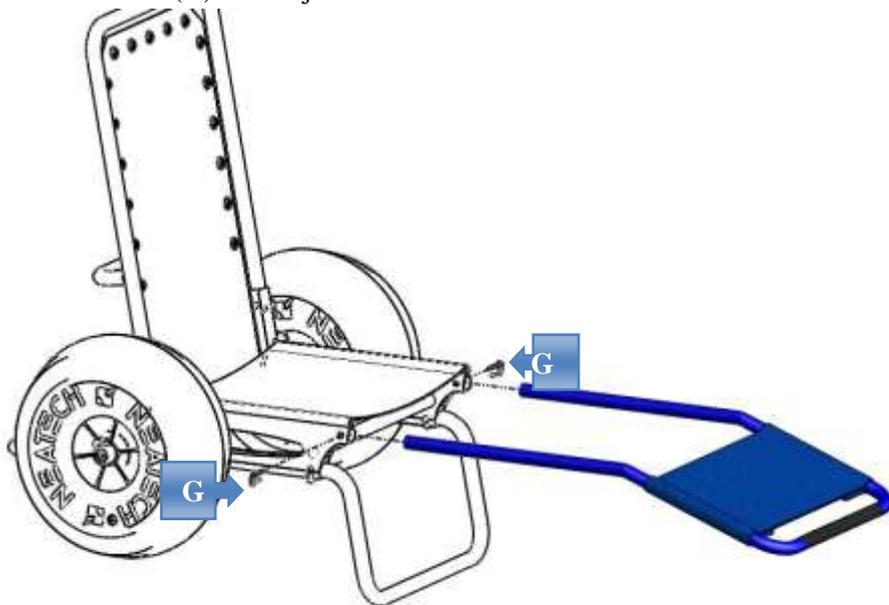
Figure 33

### WARNING

Tighten correctly the two tubes.

## Disassembly of the wheelchair

In order to disassemble the chair run the described operations in reverse.

Make sure that the locking of the wheels and tubes are trapped in their respective housings of the frame to prevent loss.

## 2.4. Transport and storage

If you don't want to use the Job for a long time, store it in a safe place free from dust and moisture.

If you want to carry the wheelchair, in order to facilitate operations, it is possible to disassemble it as described in section 2.3 and put it in the bag available as accessory. It is recommended to put the pieces of the wheelchair as in Figure 34

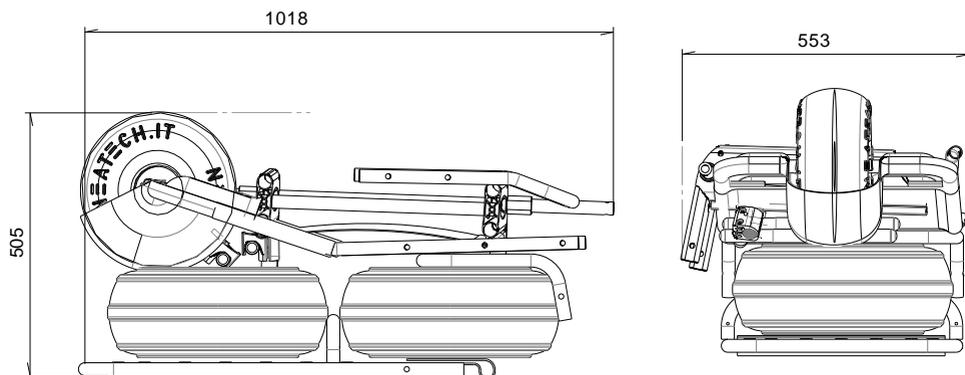
Figure 34

## 3. Adjustments

### 3.1. Legrest adjustment

To adjust the position of legrest, remove the fasteners as shown in Figure 35. Put the legrest in one of the three available positions and insert again the fasteners.

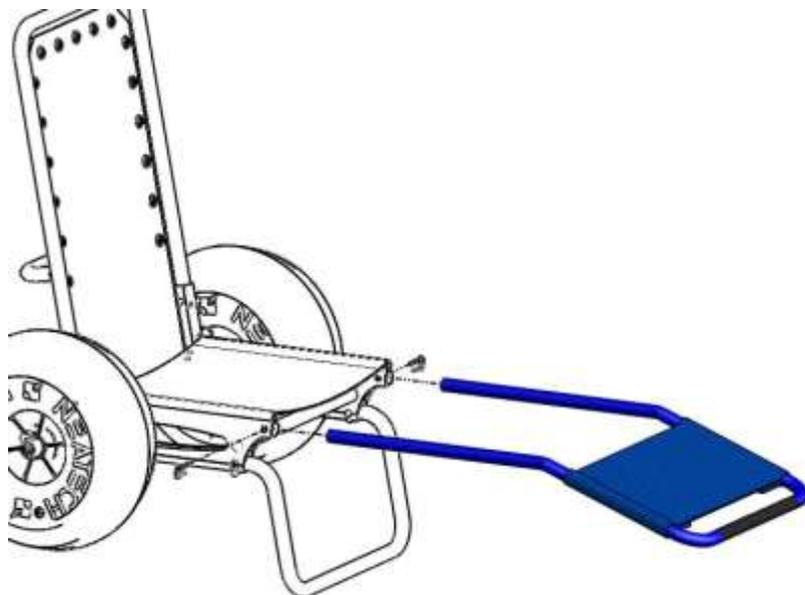
Figure 35

## 4. Use of the wheelchair

Do not use wheelchair in deep or agitated water.

Presence of an assistant is obligatory when wheelchair is into the water.

For use in safety of the wheelchair and to avoid situations of risk for the user, verify that **the circumference of the wheels is between 1380 and 1400 mm.**

If not inflate or deflate the wheel to within the recommended range.

**Figure 36**

## 4.1. Accessories

The Job can be equipped with many accessories that Neatech.it makes available to meet different customer needs.

### 4.1.1. Armrests

Assemble the armrest (rh and lh) to the backrest as shown in Figure 37. To ensure a proper connection tighten the two screws shown in Figure 37 with a 6 mm allen wrench


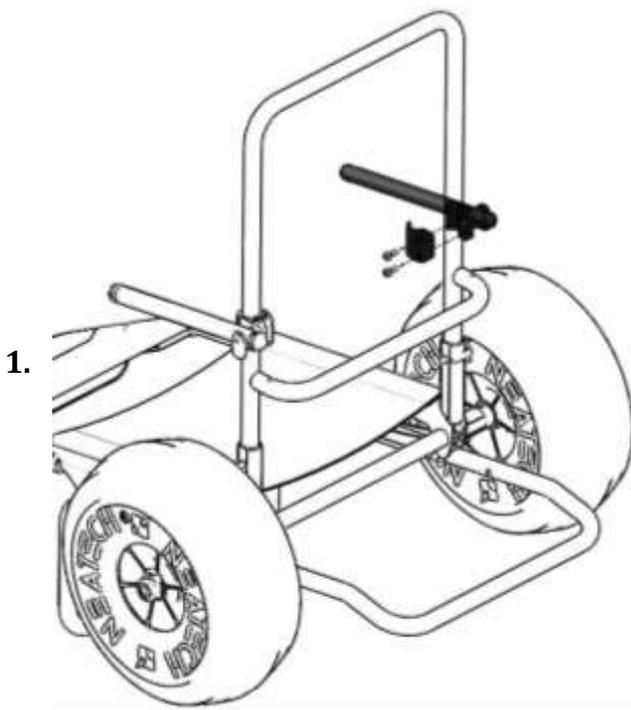
Figure 37

### 4.1.2. Third wheel kit

Please don't make this operation while the user is sitting on the wheelchair


First remove the leg bar and the front support bar simply acting on the fasteners as shown in Figure 38.

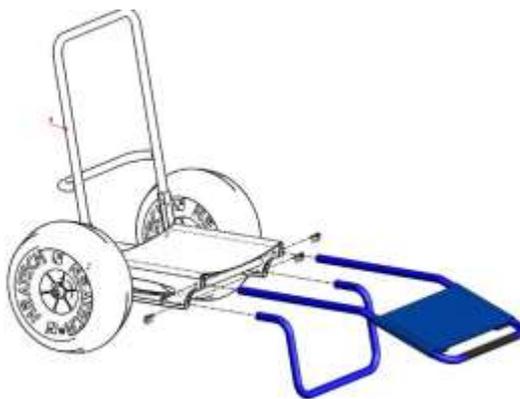**Figure 38**

So connect the third wheel kit inserting the tubes into the holes in front of the frame as shown in Figure 39. Then apply the fasteners into the appropriate holes to ensure proper connection.

The depth adjustment is the same adjustment as for the legbar. For more information see section 3.1.

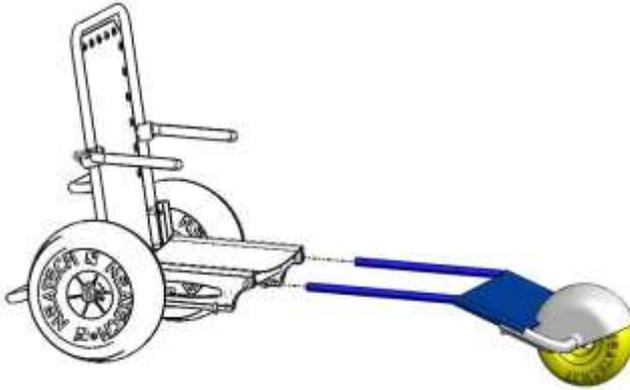**Figure 39**

Before using the third wheel kit, control that the pressure of the tire is correct.

Make sure that **the circumference of the wheel is between 81 cm and 84 cm.**

If not inflate or deflate the wheel until its circumference is in the recommended range.

### 4.1.3. Sun awning

The awning package contains:

- Bag with awning and the straps for the coupling (1)
- Supports for the awning (2)
- Awning's support bars (3)

Figure 40

From the support bars of the awnings remove the pipe cover black identifying the hole that has no outer tabs (see Figure 41).

Figure 41

Seeing Figure 42 insert the yellow flat supports in the hole just identified and then insert the support for the awning by sliding it on the backrest. Keep the media flat-head upright and facing up.

**Figure 42**

Seeing Figure 43 attach the strap to the tubular on the back of the backrest so that the hook is facing towards the outside.

**Figure 43**

Attach the strap on the back of the awning to the hook of the belt previously fixed to the tube and pull the belts till they are very stretched.

**Figure 44**

Attach the ties of the awning to the tubular of the legrest.

**Figure 45**

Adjust the tension of the tie rods acting on cord lock till the awning is stable with respect to the chair.

**Figure 46**

## 5. Maintenance

**Any work on the wheelchair must be performed by an authorized service center**

### 5.1. Maintenance and cleaning

Regular maintenance helps to keep intact the functionality and safety of JOB. Inadequate or lack of care and maintenance may cause a limitation of the warranty from the manufacturer.

To clean the chair do not use high-pressure water spray devices. For plastic and metal parts use a soft cloth dampened with mild detergent. For the upholstery, linings, seat and back covers use warm water and mild detergent. Do not use stain removers, solvents, acids, etc.

### 5.2. Controls to be performed on the wheelchair

<b>OPERATION</b>	<b>FREQUENCY</b>
<b>Tire pressure</b> For a safe use of the wheelchair check that the circumference of wheels is in the range specified in section 4 and 4.1.2.	Weekly
<b>Fasteners</b> Check for the presence of all fasteners. See section 5.5.5.	Monthly

### **5.3. Tire puncture**


In case of tire puncture please follow these instructions.

- Clean damaged surface with ethylic alcohol.
- When surface is dry; apply few drops of a glue for rigid PVC.
- Wait some minutes to let glue dry, then inflate wheels as described in this manual.

## 5.4. Specifications

Maximum user weight	125 kg
Empty weight	7 Kg - Chassis 3 Kg - Wheel
Intended use	Job is intended to all those categories of users with temporary or permanent mobility difficulties who need to make trips to the beach and other types of impervious surfaces.

### 5.4.1. Dimensions

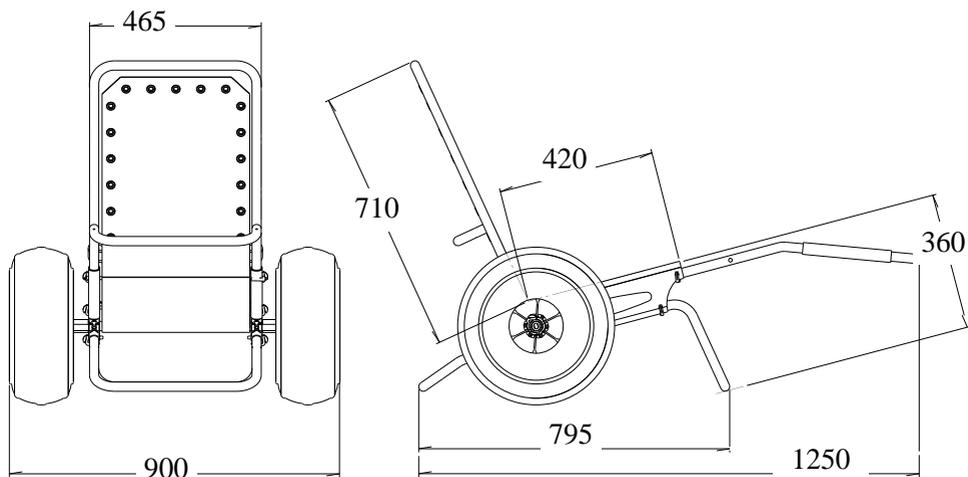Specifications may change according to the customization of each wheelchair.

## 5.5. Instructions for parts replacement

The use of spare parts or accessories not approved by the manufacturer may make the wheelchair unstable or uncontrollable.

**For each order, always contact an authorized service center.**

The final user can buy spare parts at an authorized service centers and substitute by himself.

If the user is not able to substitute himself any parts, he can contact an authorized service center.

Code	Description
<b>R105-001</b>	Wheel assembly
<b>R105-002</b>	Sling seat
<b>R105-003</b>	Sling legrest
<b>R105-004</b>	Sling backrest
<b>R105-005</b>	Fasteners kit
<b>R105-006</b>	Anti-tips bar
<b>R105-007</b>	Legbar structure
<b>R105-008</b>	Front support
<b>R105-009</b>	Cap for extruded armrest
<b>R105-010</b>	Backrest tube including sling backrest
<b>R105-011</b>	Third wheel assembly
<b>R105-012</b>	Axis of the sling seat

### 5.5.1. Wheel assembly

To substitute the wheel assembly (rh or lh) remove the fastener (F) in Figure 47 and remove the wheel.

Insert the new wheel on the axis of the frame using the appropriate location hole, previously aligned with the present hole on the axis of the frame and fully insert the locking clip (F).

Make sure that the fastener is securely attached to the hub of the wheels.

Figure 47

Code	Description
R105-001	Wheel assembly

### 5.5.2. Sling seat

To remove the sling seat, simply pull out as shown in Figure 48. Enter the new seat in the appropriate tube.

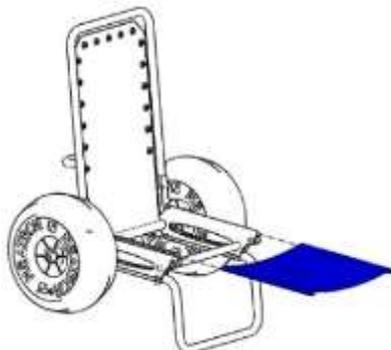
Figure 48

If necessary it is possible to replace only the rod axis simply by pulling it out of the sling seat.

Code	Description
R105-002	Sling seat
R105-012	Axis of the sling seat

### 5.5.3. Sling legrest

Remove the legrest as described in section 3.1 and remove the sling legrest. Insert the new sling legrest in the same position, insert the legrest in one of the three available positions and insert again the two fasteners in the appropriate holes.

Code	Description
R105-003	Sling legrest

### 5.5.4. Sling backrest

To replace the sling backrest release the strip on the rear part of the backrest and remove it by passing it around the axis of the wheels.


After that remove the rope, and then remove the ties that keeps it attached to the tubes.

As described enter the new sling backrest in the opposite direction.

Code	Description
R105-004	Sling backrest

### 5.5.5. Fasteners kit

If necessary you can replace the fasteners simply removing the old ones and entering the new in the same position.

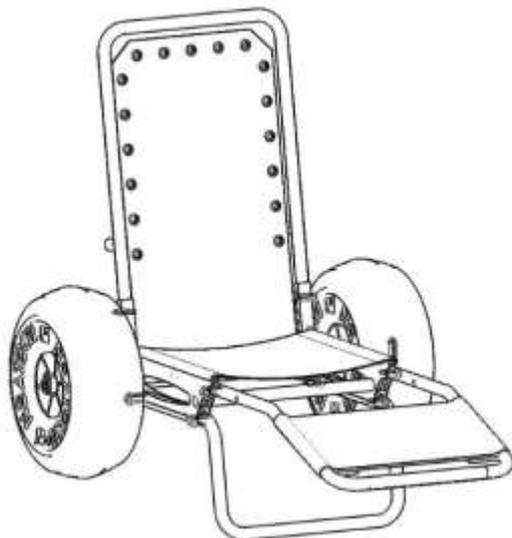**Figure 49**

The fasteners as shown in Figure 49 are located in the positions listed below:

- N°2 Backrest tube
- N°1 Right wheel
- N°1 Left wheel
- N°2 Legrest support
- N°2 Front support
- N°2 Anti-tips bar

Code	Description
R105-005	Kit of fasteners

### 5.5.6. Anti-tips bar

To change the anti-tip bar, remove the two fasteners and pull the bar as shown in Figure 50.

Insert the new bar in the same position, and be sure to properly insert the fasteners.

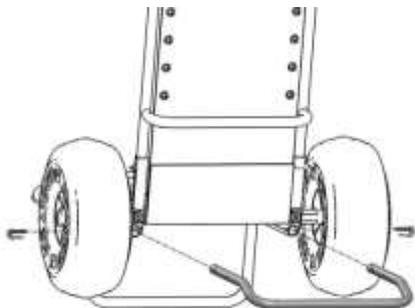
Figure 50

Code	Description
R105-006	Antip-tips bar

### 5.5.7. Legrest structure

To substitute the footrest structure remove the two fasteners and pull the bar as shown in Figure 51.

Insert the new bar in the same position, and be sure to properly insert the fasteners.

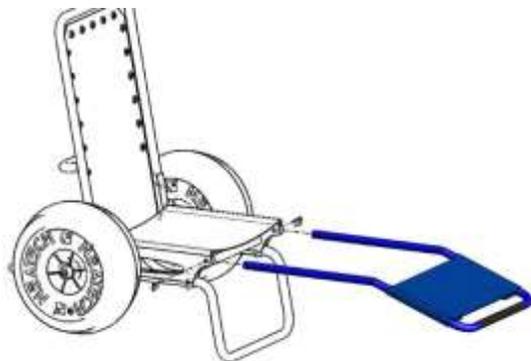
Figure 51

Code	Description
R105-007	Legrest structure

### 5.5.8. Front support

To replace the front support, remove the two fasteners and remove the bar as shown in Figure 52.

Insert the new bar in the same position, and be sure to properly insert the fasteners.

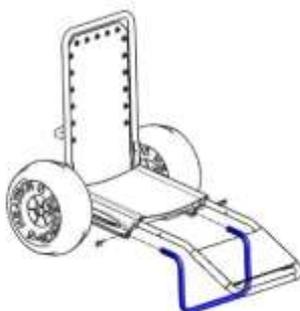
Figure 52

Code	Description
R105-008	Front support

### 5.5.9. Cap for extruded armrest

To substitute the cap for extruded armrest (rh and lh), remove the old cap as shown in Figure 53.

Insert the new one in the same position applying a light pressure.

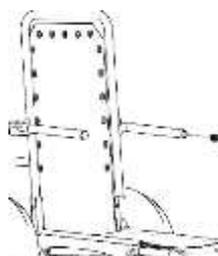
Figure 53

Code	Description
R105-009	Cap for extruded armrest

### 5.5.10. Backrest tube including sling backrest

To substitute the backrest tube comprehensive of sling backrest, remove the two fasteners and pull the bar as shown in Figure 54.

Insert the new backrest in the same position and be sure to properly insert the fasteners.

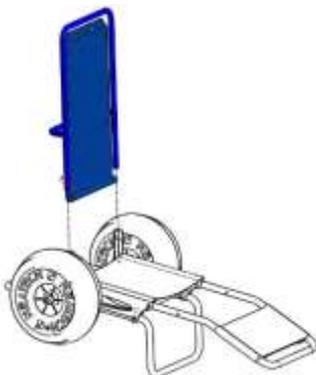
Figure 54

Code	Description
R105-010	Backrest tube including sling backrest

### 5.5.11. Third wheel assembly

To replace the third wheel assembly, unscrew the two bolts (L) shown in Figure 55 using a 5 mm allen wrench, expand the two tubes and remove the wheel.

Insert the new wheel, hook the two tubes and make sure to properly tighten the bolts previously removed.

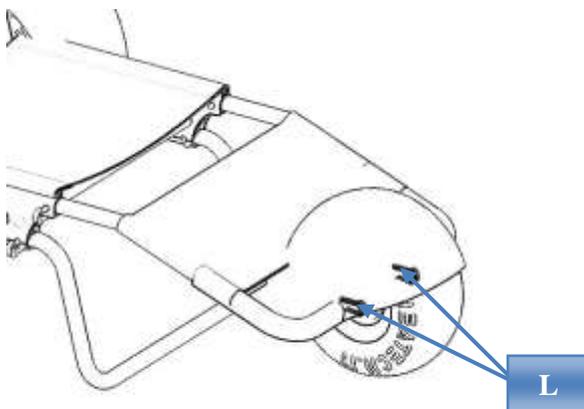
Figure 55

Code	Description
R105-011	Third wheel assembly

## 6. Warranty terms

JOB is a product globally guaranteed for 24 months. The warranty covers defects in materials or workmanship. The warranty doesn't cover parts subject to usury or damaged parts by: overload, misuse, alterations and repairs made by unauthorized third parties from Neatech.it s.r.l. .

The warranty expires in case of tampering, improper storage, unauthorized or incorrect maintenance.

### 6.1. Serial number

For any report or assistance request communicate the unique identification code on the chassis of each JOB as shown in Figure 56.

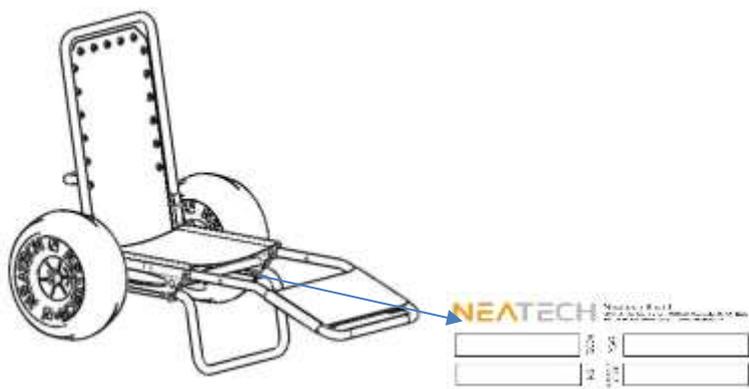
Figure 56

### 6.2. Incident reporting

If an incident occurs please contact an authorized service center. For a list of authorized service center please contact the manufacturer:

**Neatech.it srl**

4/A, A. de Curtis st.- 80040, Cercola (NA), Italy

[www.neatech.it](http://www.neatech.it) – [info@neatech.it](mailto:info@neatech.it) - +39 081 555 1946


**Neatech.it s.r.l.**

via A. De Curtis, 4/A - 80040 Cercola (NA) Italy

tel. +39 0815551946 - fax +39 0815552507

[www.neatech.it](http://www.neatech.it) - [info@neatech.it](mailto:info@neatech.it)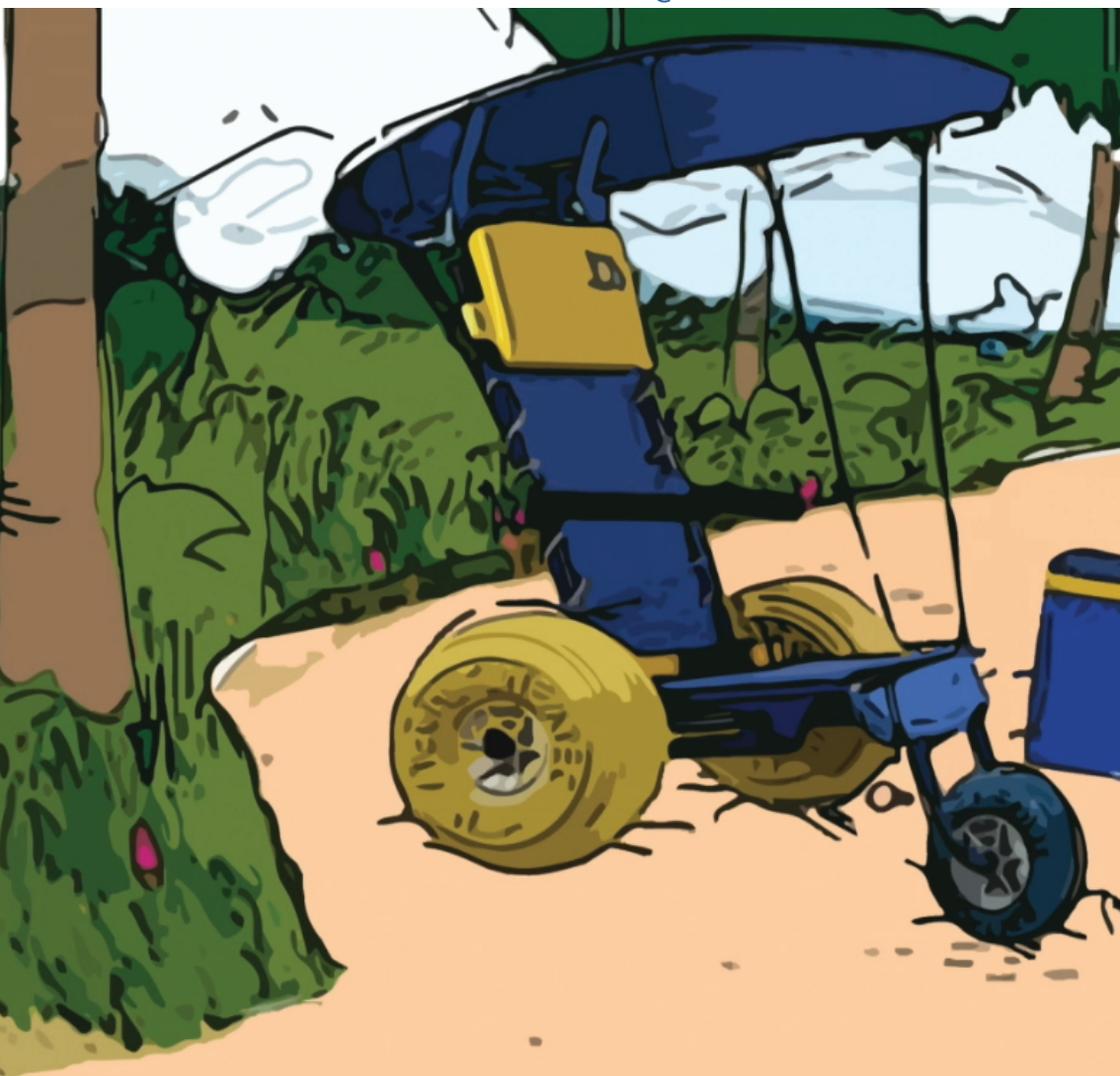

**NEATECH**™