GARANZIA

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WARRANTY

This walker has **2-year warranty** from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your walker repaired and you fulfil the conditions of warranty, send the walker to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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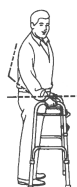
I Class Medical Device
according to ECC directive 93/42

Code RP734-735

Thank you for purchasing walker, **MOPEDIA** by Moretti manufactured to support people with movement difficulty. It is realised in lightweight anodized aluminium tubing with padded handles non-slip rubber tips casters. It is foldable and height-adjustable every 2,5 cm:

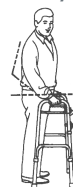
- **RP734/RP733** from 87 to 106 cm
- **RP735** from 82 to 106 cm

To adjust the height:

Push the button to adjust the height. Draw out the stick paying attention to the corresponding holes up to the desired height which should reflect user's needs. Then leave the push button to fix the stick again. Repeat described procedure for each leg, at the end, check that all four legs have the same height. Maximum endurable weight 130 kg.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.

With your arm extended down, handgrips should be at your wrist height. With your hand placed on the handgrip, your elbow should be flexed by 20° - 30°.

ATTENTION: Beware of electrical cords in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.

1. To transfer the patient from the wheelchair to the walker, be sure wheels of the wheelchair are locked.

2. To walk with the walker, lift it up and place it comfortably in front of you, making sure legs end up on the floor. Walk carefully forward, one step at a time.