MOPEDIA®

MA RP891_RP893_RP896 01_A_06 2015

**IMBRACATURE
MANUALE D'ISTRUZIONE**

**SLINGS
INSTRUCTION MANUAL**

INDICE

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I Class Medical Device
according to Directive 93/42/EEC and further modifications

RP891 Standard sling with headrest

RP893 Dorsal sling with velcro fastening

RP896 Dorsal sling with velcro fastening & pelvic traction

1. INTRODUCTION

Thank you for purchasing a MOPEDIA sling by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

2. INTENDED USE

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

The MOPEDIA by MORETTI slings are designed to transport patient with mobility disease.

3. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the antidecubitus cushions category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question ARE NOT MEASUREMENT DEVICES
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION
4. The devices in question are sold in NON STERILE packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

4. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sling
- Disinfect the item after every usage
- SERVICE LIFE- the sling use limit is defined by the wear of the parts


- Carefully read the patient lift or stand up lifter manual and follow the instruction suggested on the same
- In case of use of harnesses with a different maximum load from the one provided for the lift or stand up lifter, always use the lowest maximum load.

5. GENERAL DESCRIPTION

5.1 Sling for patient lifts

Sling for all kind of patient lift. Made in polyester (100%), especially suitable for water contact, strong and breathable, sewn with polyester thread. The border is made by a strong ribbon (100% polypropylene), which make three eyelets in every extremity to guarantee a perfect balance. The belt continuous and swen around the entire perimeter of the sling to ensure greater toughness. The seams are made using high strength polyester yarn.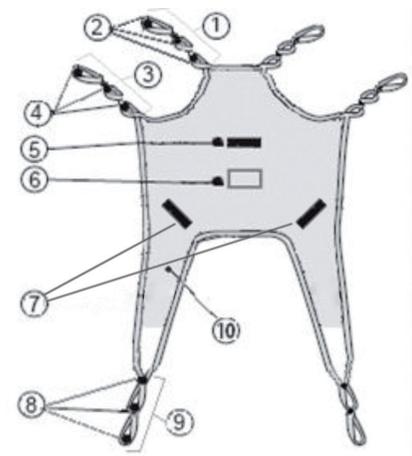

1. Upper eyelet
2. Loops
3. Intermediate eyelet
4. Loops
5. Movement handle
6. Label
7. Movement handle
8. Lower eyelet
9. Loops

REF. RP891

5.2 Sling for stand up lifter


Ensure greater toughness at each end is looped with three attachment points. with belt with Velcro for greater support when lifting the patient from a sitting to standing position.

Also available model with pelvic traction for more help in lifting (REF. RP896).

Sling for stand up lifter. Made in 100% polyester fabric (air mesh), strong and breathable, sewn with polyester thread. The border is made by a strong ribbon 100% polypropylene.


The ribbon for the border is continuous and sewn around the entire perimeter of the sling to ensure greater toughness, at each end is looped with three attachment points. Equipped with belt with Velcro for greater support when lifting the patient from a sitting to standing position.

Available also a model with pelvic traction for more help during lifting (model REF. RP896).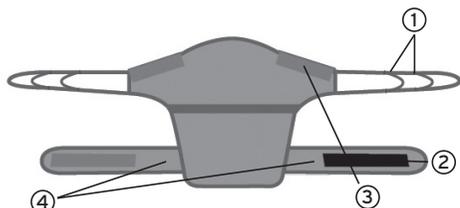

1. Loops
2. Velcro
3. Padded shoulder rest
4. Belt

REF. RP893

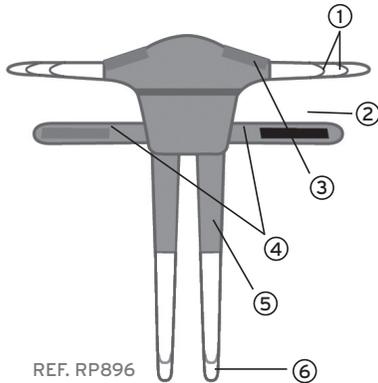REF. RP896

1. Loops
2. Velcro
3. Padded shoulder rest
4. Belt
5. Belt for pelvic traction
6. Loops

6. BEFORE USE

Always verify the sling is in perfect condition, carefully check the seams and the loops.

7. USING WARNING

- Before use, make sure the device is not damaged: the tissue, the border, the sewing and suspension loops must be intact
- Periodically check for any damage or usury, every 6 month at least, in particular if you often clean the sling
- Ensure the loops are correctly hooked to the cradle
- Store and use away from flame or source of heat
- Do not exceed the maximum load

- Evaluate the risk, in order to guarantee the exact body support unit for the patient, according to the correct dimensions, typology and shape for him
- In case of use of sling with a different maximum load from the one provided for the lift or stand up lifter, always use the lowest maximum load

8. HOW TO USE

The MOPEDIA slings by Moretti can fit all kind of patient lift, fixed, movable or stand up lifter (only RP893 - RP896 models).

9. MAINTENANCE

The devices MOPEDIA by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 6 months.

10. CLEANING AND DISINFECTION

Hand wash in water at a maximum temperature of 60°C with soap or neutral detergent.

Cautions:

- Do not use soap if not mild
- Do not use steam autoclave
- Keep far from sun light
- Do not heat

- Do not bleach
- Do not iron
- Do not use washing-machine

11. CONDITIONS OF DISPOSAL


In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sling in the appropriate disposal areas for recycling.

12. TECHNICAL FEATURES

12.1 Sling for patient lifts - REF. RP891

MATERIAL 100% Polyester
 MAX WEIGHT 180 Kg

Size	A	B	C	D
ONE SIZE	143 cm	80 cm	93 cm	35 cm


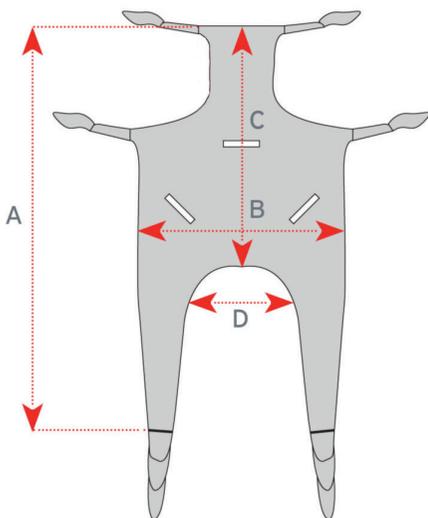
REF. RP891

12.2 Sling for stand up lifter - REF. RP893, RP896

MATERIAL 100% Polyester

DIMENSION Measures in CM

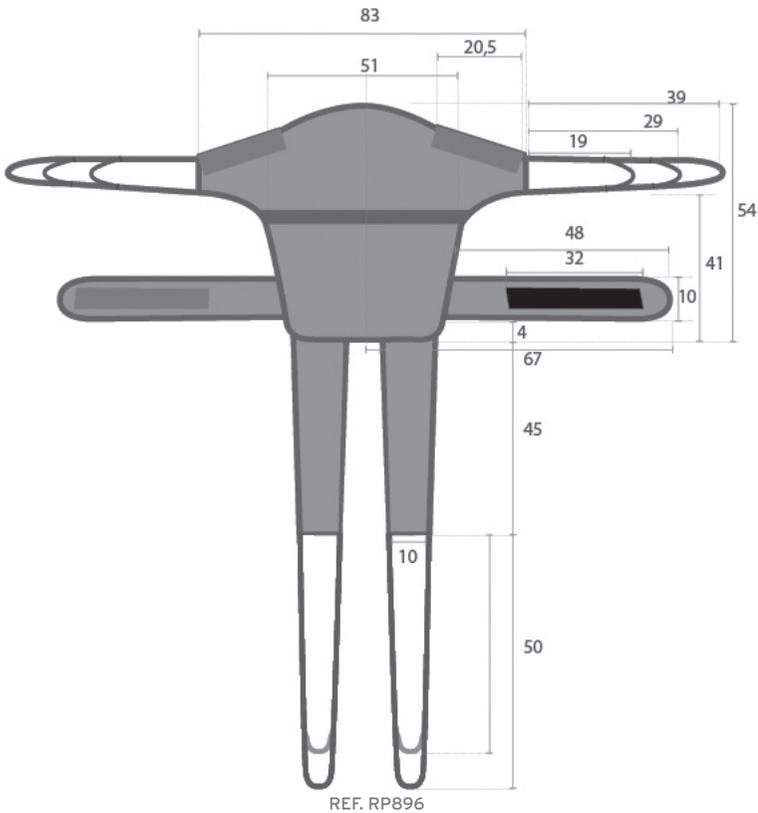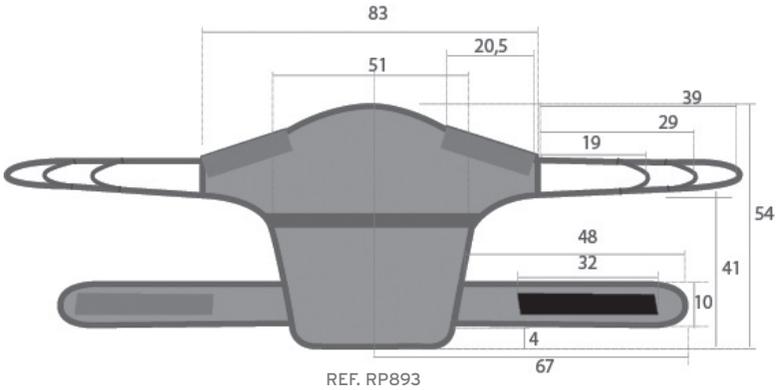13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

13.1 Exempt Clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN ITALY

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com