GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Luce di Worth

Worth light test

Test de Worth

Worth-Test

Test de Worth

Teste de Worth

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Worth light Test is used to assess a patient's flat Fusional ability.

Flat Fusional Testing is Indicated any time stereopsis falls between (50) Sec of Arc, on those patients with suspected strabismus and on preschool Children. The Worth Test should also be used when evaluating cases of reduced monocular visual acuity that does not improve with the Pinhole Test.

Procedure

1. With the best refraction correction worn by the patient, place the Anaglyph glasses over the patient's correction, with red filter over the right eye.
2. In a slightly dimmed room, turn on the flashlight and hold the Worth Test with the red dot orientated up at approximately 16" from the patient and slightly below the LOS.
3. Conduct the following Monocular Check first:
 - a. Cover the right eye; ask how many dots the patient sees. They should report 3 green.
 - b. Then Cover the left eye and ask how many dots they see now. They should report 2 red.
4. Next, Conduct the binocular test:
 - a. With both eyes uncovered ask a third time how many dots they see. If normal flat fusion is present they should report 4.

Pediatric Note: This test can reliably be conducted on preschoolers as young as 2 if they are allowed to point to the dots.

5. Abnormal responses:

- a. If the patient reports only 2 red dots under binocular conditions, this indicates that they are suppressing the left eye.
 - b. If the patient reports seeing 3 green dots under binocular conditions, then they are suppressing the right eye.
 - c. If they report 5 dots, they are diplopic. The type of diplopia can then be determined by asking which side are the green dots.
- If the green dots are located on the right, the patient has a eso deviation; to the left, an exo deviation.

- d. If the green dots are reported above or below the red dots then a vertical deviation exists. A report of the green dots above the red dots would be seen with aright hyper deviation.
- e. With a report of 6 or more dots, one should question the patient's reliability.
6. The binocular view is repeated at 5 and 10 feet, then repeat all these distances under greatly reduced light as both suppression and ocular deviation can be different under varying lighting conditions.

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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