

STOCK CODE 603387


**GP** Getein  
Biotech, Inc.

# One Step Test for SARS-CoV-2 Antigen

## (Colloidal Gold)


CE Marked

Rapid, Reliable, Early and Easy  
Detection of COVID-19

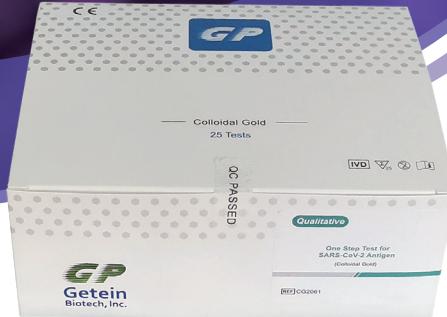


### Intended Use

The test kit is intended for qualitative detection of nucleocapsid protein antigen from SARS-CoV-2 in human nasal swab samples from patients suspected of COVID-19 infection by a healthcare provider.

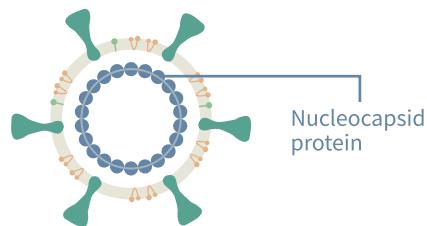

### Product Components

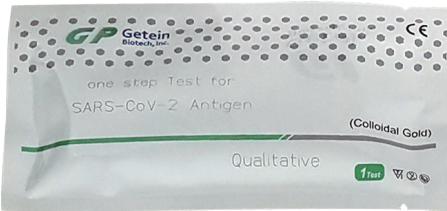


SARS-CoV-2 Antigen Test Card




Extraction Tube with Sample Extraction Solution and Tip



Sampling Swab

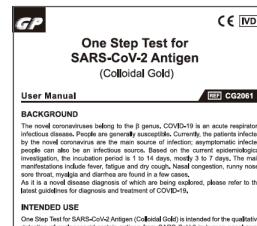

User Manual

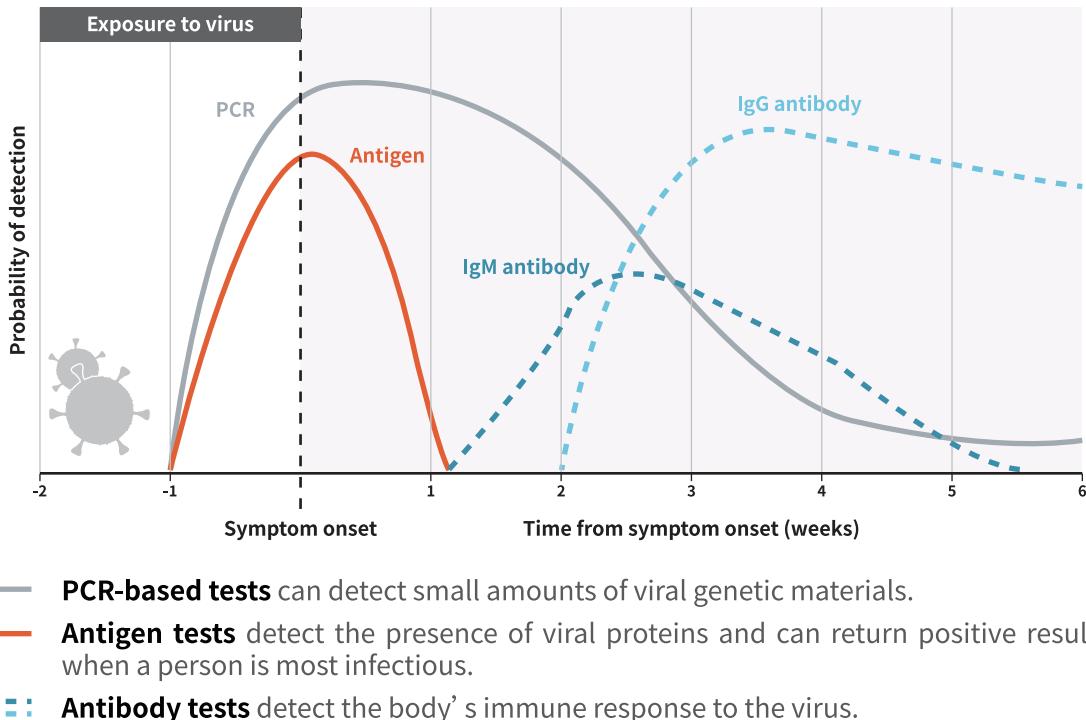

## Characteristics



Specimen type:  
nasal swab



Read test results visually.  
Do not require test equipment.



Early detection of SARS-CoV-2 infection



Rapid test  
The test result is available in 10-15 min.




Simple operation and highly-efficient test



Room temperature storage (4-30°C)

## Operation

### Nasal Swab Sampling >>

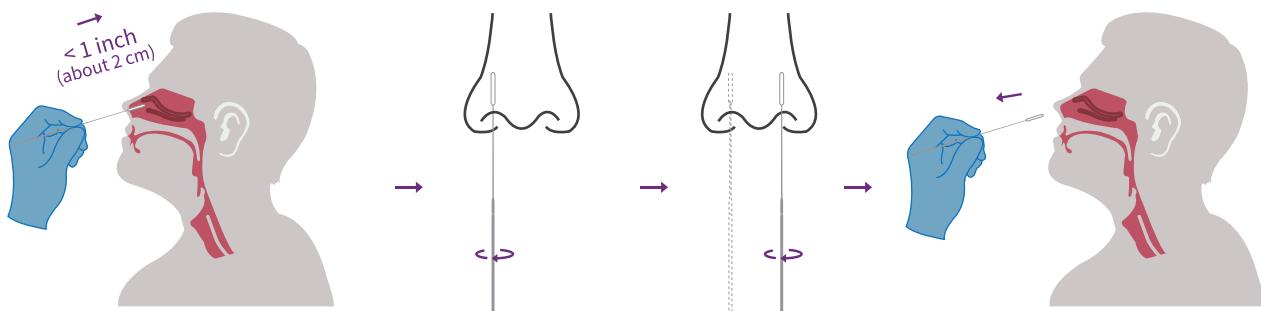


Tilt head back 70°.  
Using gentle rotation, push the swab until resistance is met at the level of the turbinates (less than one inch (about 2 cm) into the nostril).

Stay for 15-30 s,  
then rotate 3 times  
against the nasal wall

Change to  
another nostril

Remove


## Sample Pretreatment >>

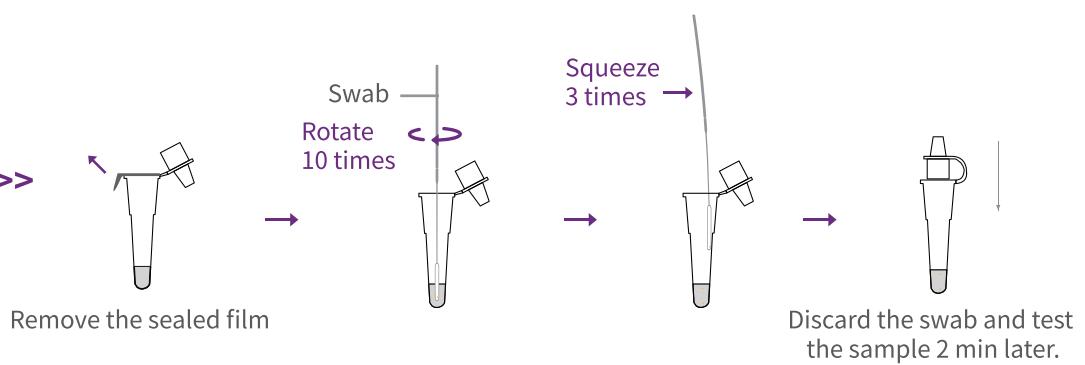

## Test >>



## Test Results

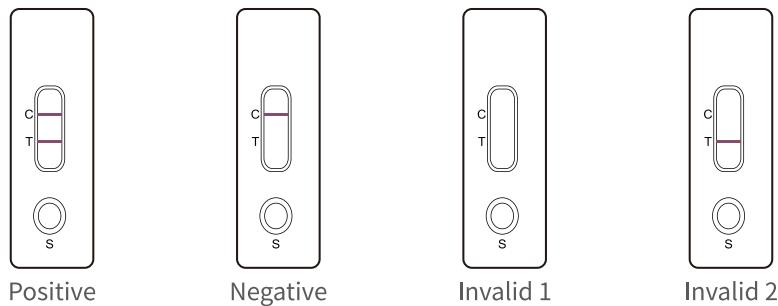

**Getein Biotech, Inc.**

Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China

Tel: +86-25-68568508/68568594

Fax: +86-25-68568500

E-mail: sales@getein.com.cn; overseas@getein.com.cn

Web: en.bio-gp.com.cn



**Lotus NL B.V.**

Add: Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands

Tel: +31644168999

Email: peter@lotusnl.com

