GIMA

Test Monofase di Gravidanza Diagnosi Precoce Cod. 29100

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GIMA Spa
Via Marconi, 1 - 20060
Gessate (MI) - Italia

Early Detection One Step Pregnancy Test Package Insert

PRINCIPLE

The Early Detection One Step Pregnancy Test is a rapid, one-step lateral flow immunoassay in strip format for the qualitative detection of human chorionic gonadotropin (hCG) in urine to aid in the detection of pregnancy. The test utilizes a combination of antibodies including a monoclonal hCG antibody to selectively detect elevated level of hCG. The assay is conducted by immersing the end of the strip in urine, and obtaining the result from the colored lines.

PRECAUTIONS

Please read all the information in this package insert before performing the test.

- Do not use after the expiration date printed on the foil pouch.
- Store in a dry place at 2-30°C or 36-86°F. Do not freeze.
- Do not use if pouch is torn or damaged.
- Keep out of the reach of children.
- For *in vitro* diagnostic use. Not to be taken internally.
- Do not open the test foil pouch until you are ready to start the test.
- For professional use

KIT CONTENT

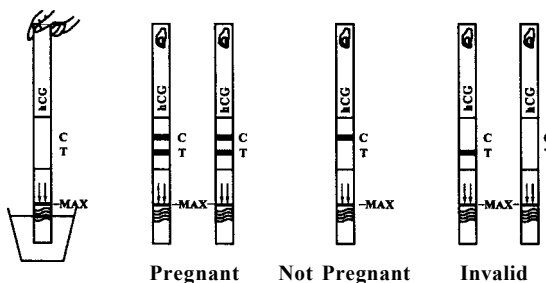
- Test strip
- Package insert

INSTRUCTION

1. Urinate into a clean, dry cup or container.
2. Remove the test strip from the foil pouch and familiarize yourself with the product.
3. With arrows pointing toward the urine, **dip the test strip vertically in the urine for a least 10 -15 seconds**. Do not pass the maximum line (MAX) on the test strip when dipping the strip onto the urine. See the illustration below.
4. Remove the test strip from the urine, place it on a non-absorbent flat surface and begin the timing.
5. As the test begin to work, you may notice a light colored flow moving across the test line region (T) and control line region (C) on the test strip. **Read the result at 3 minutes**. If no colored line appears, wait 1 minute longer. Some positive results may be observed in 1 minute or less depending on the concentration of hCG. Do not read the result after 10 minutes.

READING THE RESULTS

PREGNANT

Two distinct colored lines appear. One line should be in the control line region (C) and another line should be in the test line region (T). One line may be lighter than the other; they do not have to match. This means that you are probably pregnant.

NOT PREGNANT

One colored line appears into the control line region (C). No line appears in the test line region (T). This means that you are probably not pregnant.

INVALID

The result is invalid if no colored line appears in the control line region (C), even if a line appears in the test line region (T). You should repeat the test with a new test strip.

LIMITATIONS

There is the possibility that test may produce false results. Consult your physician before making any medical decisions.

1. Drugs which contain hCG (such as Pregnyl, Profasi, Pergonal, APL) can give a false positive result. Alcohol, oral contraceptives, painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test results.

2. A number of medical conditions other than pregnancy, including, ovarian cyst or ectopic pregnancy (pregnant outside the uterus) can cause elevated levels of hCG, which can result in a false positive result.

QUESTIONS & ANSWERS

1. How does the test work?

The Early Detection One Step Pregnancy Test detects a hormone in your urine that your body produces during pregnancy (hCG - human chorionic gonadotropin). The amount of pregnancy hormone increases as pregnancy progresses.

2. How soon after I suspect that I am pregnant can I take the test?

You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day; however, if you are pregnant, first morning urine contains the most pregnancy hormone.

3. Do I have to test with first morning urine?

Although you can test at any time of the day, your first morning urine is usually the most concentrated of the day and would have the most hCG in it.

4. How accurate is the test?

A clinical evaluation was conducted comparing the results obtained using the Early Detection One Step Pregnancy Test to another commercially available urine membrane hCG test. The consumer clinical trial included 140 urine specimens: both assays identified 80 positive and 60 negative results. The results demonstrated > 99.0% overall accuracy of the Early Detection One Step Pregnancy Test when compared to the other urine membrane hCG test.

5. How sensitive is the test?

The Early Detection One Step Pregnancy Test detects hCG in urine at a concentration of 25 mIU/mL or greater. The test has been standardized to the W.H.O. Third International Standard. The addition of LH (300 mIU/mL), FSH (1,000 mIU/mL), and TSH (1,000 μ IU/mL) to negative (0 mIU/mL hCG) and positive (25 mIU/mL hCG) specimens showed no cross-reactivity.

6. What should I do if the result shows that I am pregnant?

It means that your urine contains hCG and you are probably pregnant. See your doctor to confirm that you are pregnant and to discuss the steps you should take.

7. How do I know that the test was run properly?

The appearance of a colored line in the control line region (C) tells you that you followed the test procedure properly and the proper amount of urine was absorbed.

8. What should I do if the result shows that I am pregnant?

It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period within a week of its due date, repeat the test with a new test strip. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

You are encouraged to take the following steps to increase your chances for a healthy pregnancy and a healthy baby.

1. Use the Early Detection One Step Pregnancy Test to detect pregnancy when your period is late: You can begin better prenatal care as soon as you learn of your pregnancy.
2. If you get a positive result, it is advisable to visit your doctor immediately to begin your prenatal care.
3. Maintain a well-balanced diet, stop smoking, and reduce your intake of alcohol.

Index of Symbols

	Attention, see instructions for use
	For <i>in vitro</i> diagnostic use only
	Store between 2-30°C

	Tests per kit
	Use by
	Lot Number

	Manufacturer
	Do not reuse
	Catalog #

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